CONTROL POSITIVO EN SECO GDH-TOXINS

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DRY GDH-TOXINS POSITIVE CONTROL

INSTRUCTIONS FOR USE

The dry positive control GDH-Toxins contains a mixture of a native glutamate dehydrogenase (GDH) as well as the inactivated native Toxins A and B obtained from a *C.difficile* culture. A stabilized buffered solution is used as diluent.

The dry positive control has been prepared for use with the products Simple-Stick GDH, Simple-Stick 2A-Bdiff and Simple GDH-Toxins manufactured by OPERON S.A. Use with any device other than those indicated and the result obtained is EXCLUSIVE RESPONSIBILITY OF THE USER. OPERON S.A. is not responsible for it.

Materials included in the kit

The positive control GDH-Toxins JUST CONSISTS OF THE SWAB. Any other material indicated in the following explanations has to be provided by the user.

Procedure for use with Simple-Stick GDH and Simple-Stick 2A-Bdiff:

- Remove the swab from its container just when it is to be used.
- Add 500 µl of the sample diluent buffer to the 1.5 ml eppendorf tube included in the Simple-Stick GDH and Simple-Stick 2A-Bdiff kits.
- Insert the swab into the eppendorf tube and moisten it well with the buffer, rotating the swab against the walls of the tube for at least 1 minute. Keeping the swab inside the tube, remove it from the liquid and extract the maximum amount of the impregnated liquid by pressing it against the walls of the tube. Discard the swab.

▪ For use with the Stick GDH and Stick 2A-Bdiff strip:

- Add the resuspended Positive Control GDH-Toxins to a test tube with a non-graduated disposable plastic pipette. Both components are included in the Stick GDH and Stick 2A-Bdiff kits. Add the necessary amount so that the stick is soaked in the liquid but does not exceed the arrows.

- Remove the test from its container and insert it into the test tube with the arrows down. Throw away the silica gel.

- Incubate at room temperature 10 or 15 minutes depending on whether the Stick GDH or Stick 2A-Bdiff is being used respectively. Read the result in the central area of the test.

▪ For use with the Simple GDH and Simple 2A-Bdiff strip:

- Remove the test from its container. Discard the silica gel
- Add 5 drops of the GDH-Toxins resuspended Positive Control in the circular area of the sample application window with a non-graduated disposable plastic pipette (included in the Simple GDH and Simple 2A-Bdiff kits).

- Incubate at room 10 or 15 minutes depending on whether the Simple GDH or Simple 2A-Bdiff is being used respectively. Read the result in the central area of the test.

In the GDH test, a **BLUE** line corresponding to the control band and a **RED** line below

the control band corresponding to the GDH test band must be observed.

In the 2A-Bdiff test, a **BLUE** line corresponding to the control band, a **RED** line below the control band corresponding to Toxin A and another **RED** line above the control band corresponding to Toxin B.

Procedure for use with Simple GDH-Toxins:

- Remove the swab from its container just when it is to be used.
- Unscrew the cap of the individual vial (included in the Simple GDH-Toxins kit) and insert the swab, moistening it well in the sample diluent buffer. Rotate the swab against the walls of the vial to release the antigen. Squeeze the impregnated liquid by pressing the swab against the walls of the vial.
- Remove the swab, close the vial well with its cap and shake vigorously to ensure a homogeneous mixture.
- Remove the reaction device from the aluminum bag. Discard the desiccant sachet since it only serves to preserve the moisture test and is not used to perform the test.
- Break the upper end of the vial cap.
- Add 4 drops in the sample addition zone of each of the test strips for the detection of GDH-Toxins.
- Wait at room temperature for 15 minutes. Read and interpret the results.

On the one hand, a **BLUE** line that corresponds to the control band, as well as a **RED** line below the control band that corresponds to the GDH band appear in the reaction strip for the detection of GDH.

On the other hand, a **BLUE** line that corresponds to the control band, as well as a **RED** line below the control band that corresponds to Toxin B and another **RED** line below the Toxin B band that corresponds to Toxin A should appear in the reaction strip for the detection of Toxins.

The stability of both controls is 24 months if stored between +2°C and +30°C. Meet ALWAYS the Good Laboratory Practises when manipulating the controls.

The GDH-Toxins positive control is negative for the usual infectious agents, since it does not come from a human cell culture and therefore is free of any human blood element. Please read the product safety sheet.

 Rev. 01 – 04.10.2019

Expiry date

Manufactured by

Lot number

Reac package insert

For in vitro diagnostic use

This product fulfills the requirements of Directive 98/79/EC on in vitro diagnostic medical devices.

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OPERON, S.A. Camino del Plano,19 E-50410 Cuarte de Huerva (Zaragoza) Spain + 34976 503597